


DRINK SPIKE TEST



Supplied by authorised distributor:



www.1standbowery.co.uk

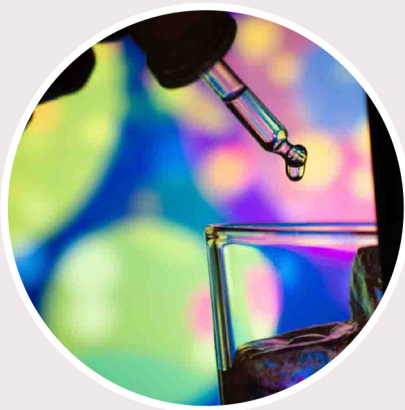

How to use it

Take a drop of the drink with a straw and place it on both the testing pads. The colours of the pads are pink and yellow. If the colour changes to BLUE and ORANGE, the drink has been spiked. CYD can also be tested in vomit. Don't forget to record the result (negative or positive) using the QR code. Follow the instructions in the app and refer to your venue's policies.

CYD has a 97% detection rate with 99% accuracy for detecting GHB and Ketamine